# MERIT MISER<sup>®</sup>

## CONTRAST MANAGEMENT SYSTEM

U.S. Patent Number 5,569,208

CE 0123

### I N S T R U C T I O N S F O R U S E :

**DESCRIPTION:** Contrast delivery system.

**INDICATIONS AND USAGE:** The Miser Contrast Management System is recommended for the controlled administration of contrast.

**CAUTION:** Federal (USA) law restricts this device to use by or on the order of a physician. This device is intended for single patient use. Read instructions prior to use.

**INSTRUCTIONS FOR USE:** Inspect the device prior to use to verify that no damage has occurred during shipping.

- 1: Turn stopcock to "off" position on spike assembly. Attach contrast spike assembly to contrast bottle/bag.
- 2: Attach burette assembly to contrast spike assembly.
- 3: Attach fixed male luer of burette assembly to manifold port.
- 4: Turn spike assembly stopcock "on" to allow flow from contrast bottle/bag into burette.
- 5: Fill burette to the Fill Level (10mL) and stop contrast flow by turning the 1-way stopcock on the top of the burette.

(NOTE: The contrast in the burette will refill to approximately the same level until the contrast bottle/bag is depleted.)

(NOTE: Never inject greater than 12mL (Max Level) back up into the burette.)

6: To maximize contrast savings, turn the 3-way spike stopcock "off" to the contrast bottle/bag. This step should take place prior to the last two contrast injections.

(NOTE: Step #6 allows the clinician to continue drawing contrast and empty the burette. The blue floating disc prevents air from entering the line as the remaining contrast in the burette is depleted.)

7: Open the 3-way stopcock below the burette "to air." This allows the remaining contrast in the proximal tubing to be aspirated.

(CAUTION: Ensure that air in the tubing is not drawn through the manifold.)

8: Remove the Merit Miser from the Contrast Management Spike and discard at the conclusion of the case. Attach male port protector to distal end of the Contrast Management Spike 3-way stopcock.

(CAUTION: Do not use Miser Contrast Management System Spike Assembly for more than 6 hours.)

EC MFR

Merit Medical Systems, Inc.  
South Jordan, Utah 84095  
1-801-253-1600  
U.S.A. Customer Service  
1-800-356-3748

EC REP

Merit Medical Ireland Ltd  
Parkmore Business Park West  
Galway, Ireland

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