KanguruWeb™

Every patient deserves your best care

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
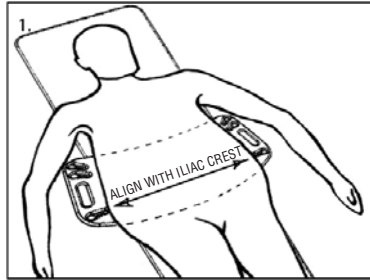
INTENDED USE

The disposable web is used to lift and support the abdominal pannus away from the femoral artery/groin area to assist in visualization and unimpeded access to the femoral artery for vascular access.

BASE UNIT POSITIONING

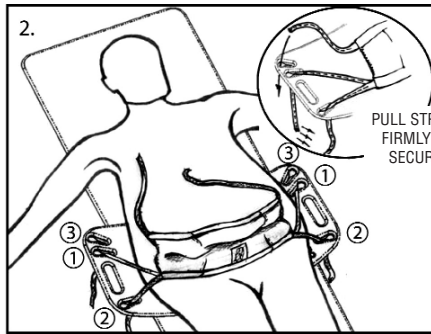1. Position base unit on the cath table, under the mattress pad, prior to patient transfer. Orient the base unit so the set of top 2 cleats is toward the patient's head. Position the single cleat set toward the patient's feet just proximal to the patient's iliac crest. The base unit must be positioned at or above the level of the iliac crest to achieve proper pannus retraction.


DISPOSABLE WEB DEPLOYMENT

2a Place disposable web on patient's abdomen and orient the web with the 2 bottom straps toward the patient's feet. Locate and thread the 2 bottom straps through the bottom and middle cleats of the base unit. Locate and thread the top (dark blue) strap through the top cleat.

2b The middle (light blue) lifting straps are the first to secure. Hold bottom of web on crease of patient's abdomen with one hand and pull on the middle strap with the other hand until the pannus is retracted. Secure middle strap by pulling it into the narrow part of the cleat, repeat on other side.

2c Secure bottom (white) securing straps next, using the same method.

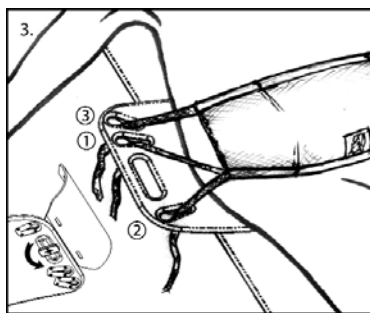2d Secure top (dark blue) stabilizing straps last, using the same method.

LOCKABLE ARM BOARD SECUREMENT

If using the KanguruWeb™ arm rests, insert arm rest in handle of the KanguruWeb™ base unit and turn knob underneath to lock in place. If using other arm rests, slide arm rest between the base unit and the procedure table after you have implemented the web.

REUSABLE DEVICE INFORMATION – BASE UNIT AND ARMBOARDS

Reusable. Clean using approved disinfectant. Do not sterilize, do not autoclave.

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Merit Medical Systems, Inc. South Jordan, Utah 84095 • U.S.A. 1-801-253-1600
U.S.A. Customer Service 1-800-356-3748
Merit Medical Ireland Ltd, Parkmore Business Park West, Galway, Ireland