

NOTE: Follow your standard facility protocol for care and cannulation.

1) Identify the Patient

- Patient Identification Wallet Card
- HeRO Graft patients will typically have 3 incision sites (see circled areas below). **LOCATE** these incision sites

2) Timing of Cannulation



Assess the HeRO Graft to evaluate for first cannulation approximately two weeks post implant per KDOQI guidelines.

3) Assessment

- **LOOK** for a uniform sized graft with NO irregularities or aneurysm formations.
- **LISTEN** for low pitch, continuous diastolic and systolic flow. HeRO Graft bruit may be soft due to absence of a venous anastomosis.
- **FEEL** the thrill; strongest at the arterial anastomosis. May be less prominent due to absence of a venous anastomosis.

4) Cannulation

- A light tourniquet may be used to dilate the graft.
- If cannulating toward the arterial anastomosis incision, stay at least the length of the fistula needle from the incision site.
- **NEVER** cannulate the Venous Outflow Component. Cannulate 3" (8 cm) from the Connector incision site to avoid damage to the graft rings.

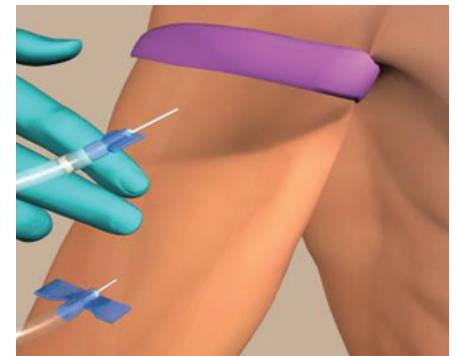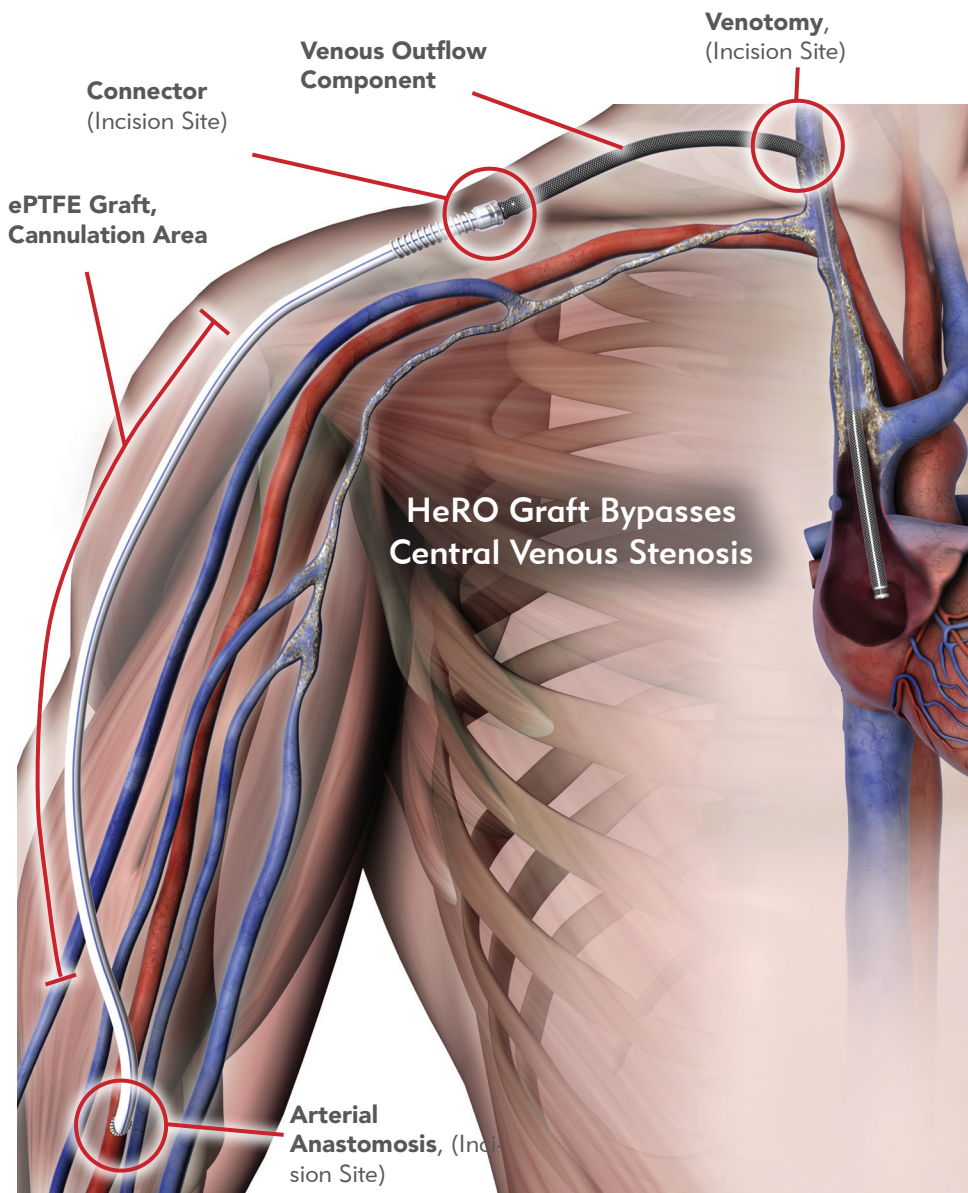

- Avoid the use of fistula clamps for hemostasis.

NOTE: Remove bridging catheter as soon as possible after successful HeRO Graft cannulation.

Learn more at MeritEMEA.com/hero



Understand. Innovate. Deliver.™

Merit Medical Systems, Inc.
1600 West Merit Parkway
South Jordan, Utah 84095
1.801.208.4300
1.800.35.MERIT

Merit Medical Europe, Middle East,
& Africa (EMEA)
Amerikalaan 42, 6199 AE Maastricht-Airport
The Netherlands
+31 43 358 82 22

Merit Medical Ireland Ltd.
Parkmore Business Park West
Galway, Ireland
+353 (0) 91 703 733