


STABILIT[®]

Vertebral Augmentation System




ACCESS + NAVIGATION + STABILIZATION

Merit Medical's intuitive platform is designed to treat vertebral compression fractures, and can provide rapid and lasting pain relief¹ with the most advanced targeted therapies. The StabiliT Vertebral Augmentation System is a minimally invasive procedure which provides physicians with a simple device that enables control over access, navigation, cement delivery, and radiation exposure.

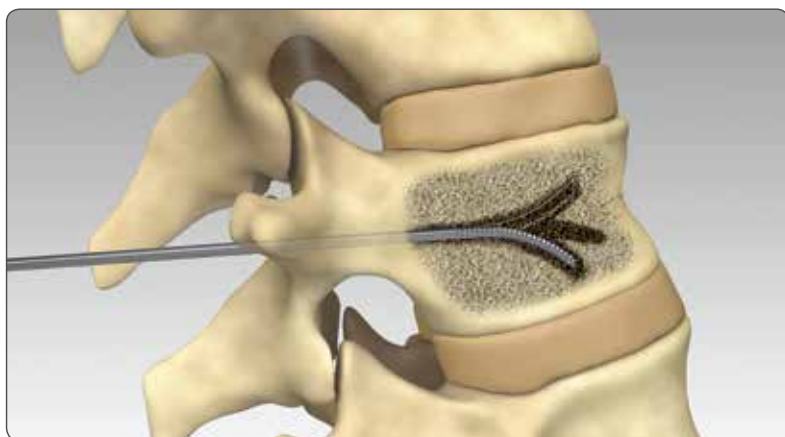


The PowerCURVE[®] Navigating Osteotome enables targeted vertebral access across the midline via a unipedicular approach.*

Targeted cavity creation spares cancellous bone as it creates preferential pathways for the flow of ultra-high viscosity StabiliT ER² Bone Cement.

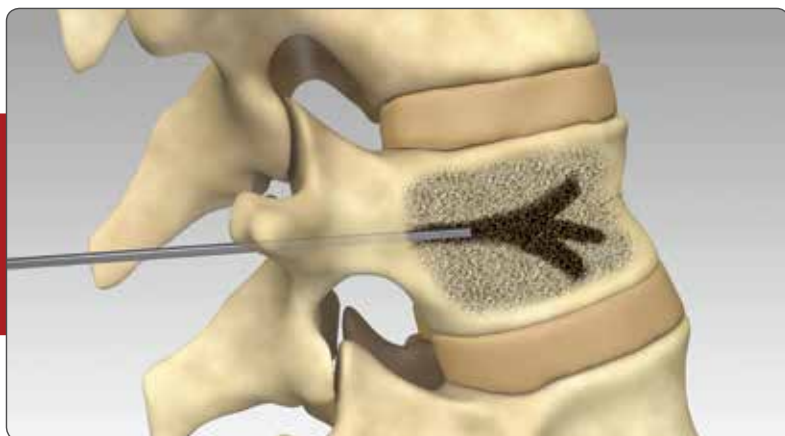

Targeted pathways maximize the exposed surface area available for cement interdigitation, resulting in a predictable fill and improved mechanical strength.

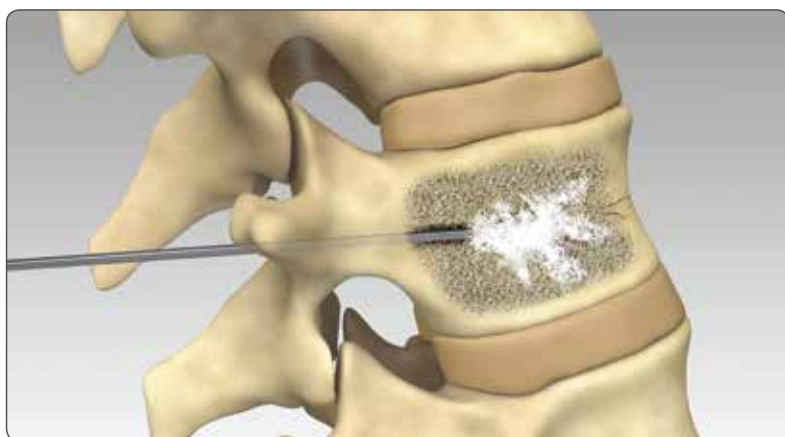

1. Pflugmacher, R. (2012, February). [Comparison of clinical and radiological data in the treatment of patients with osteoporotic vertebral compression fractures using radiofrequency kyphoplasty or balloon kyphoplasty]. Retrieved January 25, 2017, from <https://www.ncbi.nlm.nih.gov/pubmed/21993914>

* Data on file.

INTELLIGENT ENERGY™

Extended viscosity control


The MultiPlex II Controller with Variable Viscosity modulates the application of radio-frequency energy and controls the consistent delivery rate of bone cement.



Viscosity adjusted in real-time

The exclusive cement viscosity algorithm continuously monitors cement viscosity and adjusts polymerization of ER² bone cement, by adjusting RF energy delivery to provide consistent and predictable viscosity. As bone cement passes through the Activation Element, RF energy accelerates polymerization to increase the viscosity of the cement prior to delivery into the vertebral body. The MultiPlex II Controller responds to changing conditions in real time in order to maintain control over cement viscosity and delivery.

The Variable Viscosity feature allows the user to adjust cement viscosity intraoperatively (High \rightleftharpoons Low) to tailor cement viscosity to user's preference and case specific requirements.

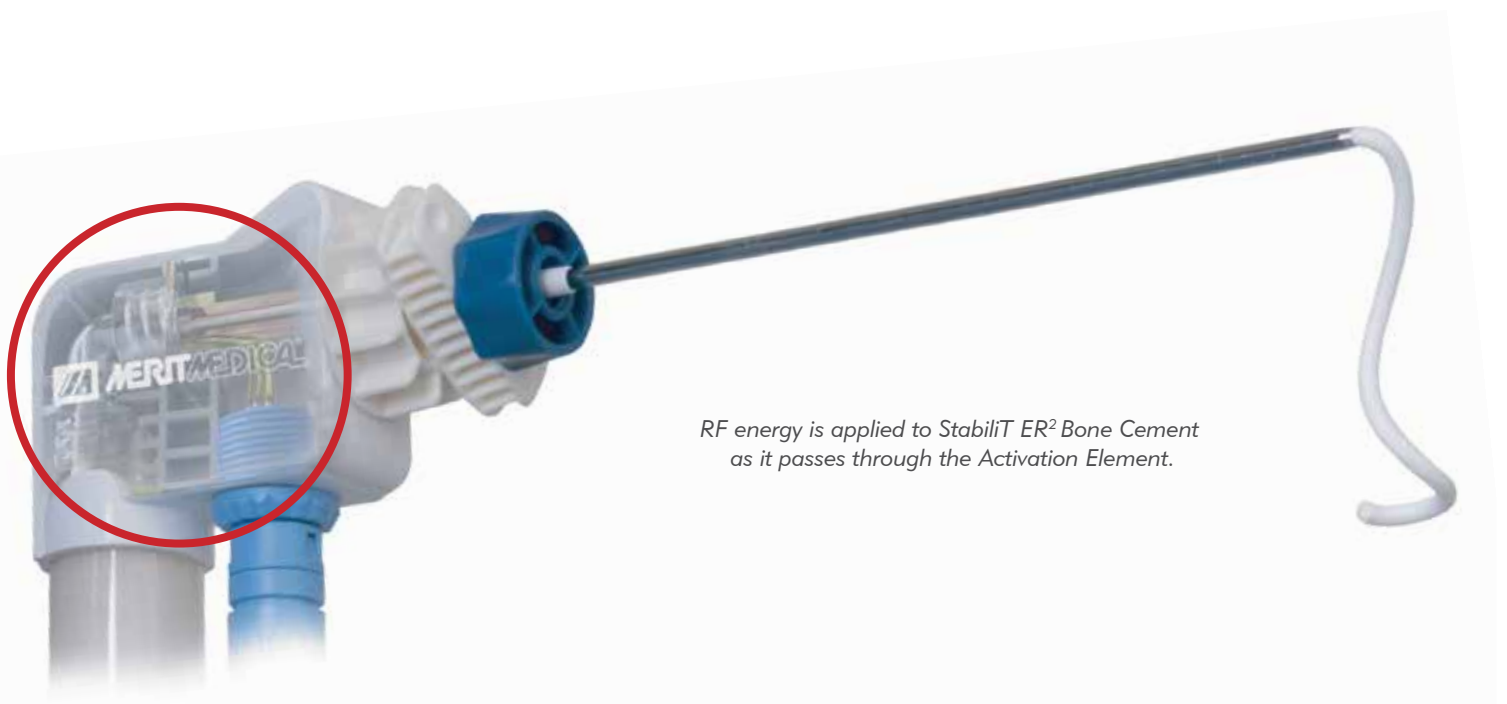

RF energy is applied to StabilIT ER² Bone Cement as it passes through the Activation Element.

BONE CEMENT

Energy-responsive (ER)

The StabiliT System utilizes proprietary energy-responsive StabiliT ER² bone cement that provides consistent, ultra-high viscosity properties over an extended working time.

Superior interdigitation

Vertebral augmentation with ultra-high viscosity StabiliT ER² bone cement delivers superior interdigitation.




Extended working time




StabiliT ER² Bone Cement has a working time of at least 35 minutes for a longer, more controlled delivery.*

Reduced extravasation with vertebral augmentation (VA)

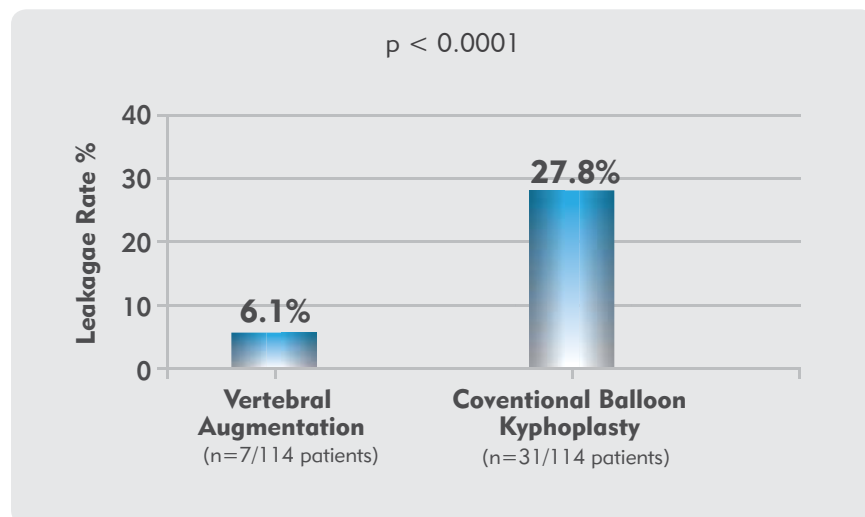

Studies suggest that fracture morphology, cement viscosity, and the rate of cement injection may influence the likelihood of cement extravasation during vertebral augmentation.^{1,2}

Reduced radiation exposure

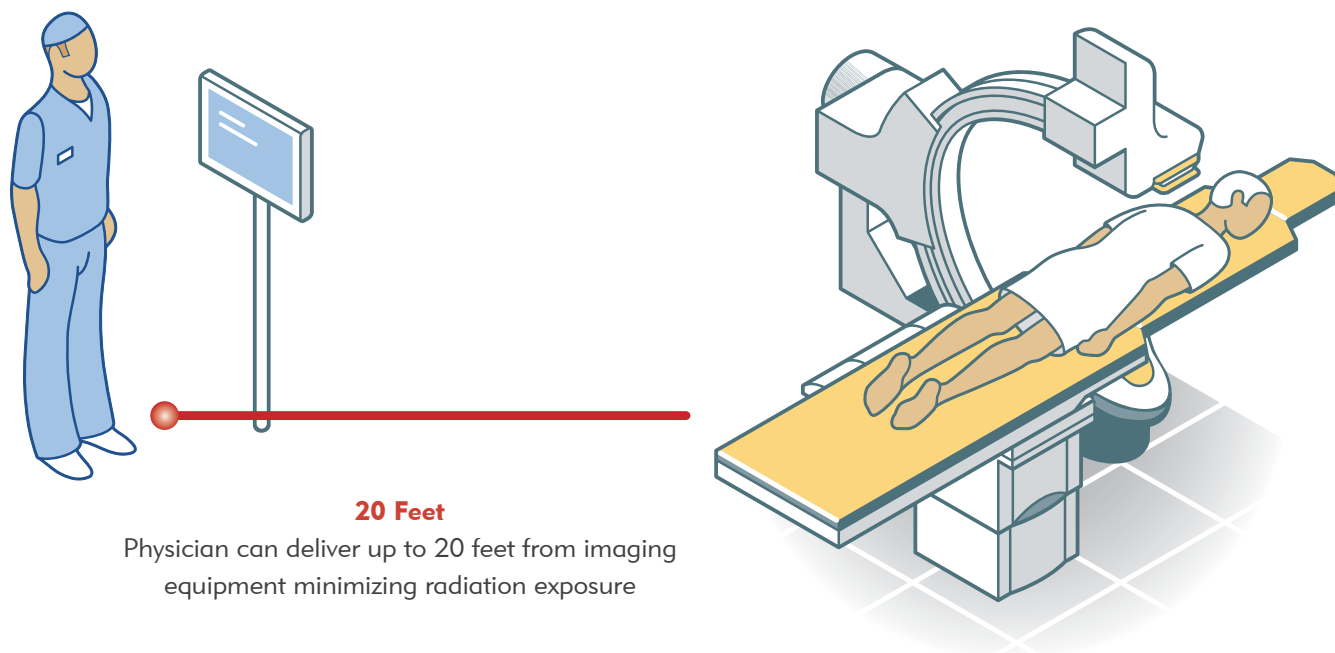

The intensity of radiation exposure dissipates exponentially as the distance from the radiation source increases. The StabiliT System Hand Switch Cable allows a physician to work up to 20-feet away from the source of radiation during StabiliT ER² Bone Cement delivery.

1. Pflugmacher, R. (2012, February). [Comparison of clinical and radiological data in the treatment of patients with osteoporotic vertebral compression fractures using radiofrequency kyphoplasty or balloon kyphoplasty]. Retrieved January 25, 2017, from <https://www.ncbi.nlm.nih.gov/pubmed/21993914>
2. Lador, R., Dreiangel, N., Ben-Galim, P.J., Hipp, J.A. (2010). A pictorial classification atlas of cement extravasation with vertebral augmentation. *The Spine Journal*, 10(2010), 1118-1127. doi: 10.1016/j.spinee.2010.09.020.

STABILIT Vertebral Augmentation System Indications and Risks

The StabiliT Vertebral Augmentation System is intended for percutaneous delivery of StabiliT Bone Cement in vertebral augmentation (kyphoplasty) procedures in the treatment of pathological fractures of the vertebrae. Painful vertebral compression fractures may result from osteoporosis, benign lesions (hemangioma), and malignant lesions (metastatic cancers, myeloma).

As with most surgical procedures, vertebral augmentation (VA) procedures using the StabiliT System involve risks, some of which may be serious or even fatal. Patients undergoing VA are subject to risks associated with surgery including, but not limited to, complications from anesthesia/sedation as well as bleeding, bruising, pain, infection, blood clots, myocardial infarction, cardiac arrest, stroke, pulmonary embolism, cardiac embolism, as well as damage to normal tissue, nerves, the spinal cord or other structures, which could result in injuries, including paralysis. Not every patient is a good candidate for VA. The StabiliT System is contraindicated for patients with coagulation disorders, severe pulmonary insufficiency, and certain spinal conditions, or patients who are sensitive to components of StabiliT Bone Cement. Please review the Instructions for Use for a more detailed description of risks and contraindications.

Catalog No.	Description
3610	MultiPlex II Controller with Variable Viscosity

KIT CONFIGURATIONS

2003-01	StabiliT First Fracture Kit with PowerCURVE (Long)
3353-01	StabiliT First Fracture Kit with PowerCURVE (Short)

A LA CARTE (SHORT/LONG)

1488/1467	StabiliT Introducer (Bevel)
1493/1472	StabiliT Introducer (Diamond)
0975/1426	Locking Delivery Cannula
1011/1545	VertecoR [®] StraightLine Cement Staging Osteotome
2011S/2011L	PowerCURVE Navigating Osteotome
2224	VertecoR Bone Drill
1688	StabiliT ER ² Bone Cement and Saturate Mixing System
1402	Hydraulic Assembly
1155	Activation Element (AE)
0860	AE Cable
0856	Hand Switch Cable



Understand. Innovate. Deliver.™

Merit Medical Systems, Inc.
1600 West Merit Parkway
South Jordan, Utah 84095
1.801.208.4300
1.800.35.MERIT

Merit Medical Europe, Middle East, & Africa (EMEA)
Amerikalaan 42, 6199 AE Maastricht-Airport
The Netherlands
+31 43 358 82 22

Merit Medical Ireland Ltd.
Parkmore Business Park West
Galway, Ireland
+353 (0) 91 703 733

merit.com

Australia
+61 (0) 3 8373 4943

Austria
0800 295 374

Belgium
0800 72 906 (Dutch)
0800 73 172 (Français)

Brazil
+55 11 4561 - 0062

Canada
1.800.364.4370

China
+86-10 8561 0788

Denmark
80 88 00 24

Finland
0800 770 586

France
0800 91 60 30

Germany
0800 182 0871

Hong Kong
+852-2207 6888

India
+91-80-41223376

Ireland (Republic)
1800 553 163

Italy
800 897 005

Korea
+8210 2205 0707

Luxembourg
8002 25 22

Netherlands
0800 022 81 84

Norway
800 11629

Portugal
308 801 034

Russia
+7 495 221 89 02

Spain
+34 911238406

Sweden
020 792 445

Switzerland
(Deutsch)
+41 22 518 02 30

(Français)
+41 22 518 02 52

(Italiano)
+41 22 518 00 35

UK
0800 973 115