en

Warning: Test firing the needle may damage the cutting cannula and/or stylet tip. Keep fingers or other objects from in front of the needle as it will advance 2.5 cm very rapidly when activated. Never test fire the instrument in an unsafe direction.

Step 1: Preparation of Achieve for Use

To charge the Achieve biopsy device, simply pull back on the charging arm and release twice, first to charge the cutting cannula, the second to charge the inner stylet.

Step 2: Position the Needle

Position the needle tip at or near the edge of the region to be sampled.

Step 3: Fire the Instrument

Two firing sequence methods are available once the Achieve biopsy device is charged.

- Delay Mode–Depression of the D button releases the inner stylet into the sample area. The operator uses this delay firing method to verify the position of the sample notch in the target area prior to depressing the A button which activates the cutting cannula, capturing the tissue sample.
- Automatic Mode–The operator can choose the fully sequential mode, omitting the delay feature, simply by depressing the A button first. This releases the stylet and cutting cannula in sequence, capturing the tissue sample.

A=Automatic Mode
D=Delayed Mode (Stylet Only)

Step 4: Biopsy Specimen

To retrieve the tissue sample, hold the instrument with the A and D firing buttons facing up, and pull back and release the charging arm once. This action will retract the outer cutting cannula exposing the stylet notch with the tissue sample.

Step 5: Obtaining Multiple Samples

After the tissue sample is removed, additional samples from the same patient may be obtained by pulling the stylet into the target area second time, which will retract the stylet within the cutting cannula, placing the Achieve biopsy device once again in a fully charged position. Repeat Steps 3-5.

In the automatic mode, the Achieve automatic biopsy device is designed to fire the cannula and stylet in rapid succession. Please exercise caution before proceeding as the device will advance 2.5 cm when activated.

Caution: For single use only. Re-use may result in a non-functional product or contribute to gross contamination.

Only clinicians who have been properly trained in this procedure, including patient preparation, sample preservation and the use of image guidance for percutaneous biopsy, should use this device.

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Achieve, CareFusion and the CareFusion logo are trademarks or registered trademarks of CareFusion Corporation, or one of its affiliates.

© 2010, 2011, 2013, 2014, 2017, CareFusion Corporation or one of its affiliates. All rights reserved.

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Made in Dominican Republic by CareFusion DR 203 LTD
Sponsored by
CareFusion Australia 316 Pty. Ltd.
CareFusion New Zealand 313 Ltd.
362-23302 • 2017-Q2

 **CareFusion**

DCN 60710
Vernon Hills, IL
David Knuth
02-07-17
362-23302

Proofed by: _____	Date: _____
Dimensions checked: _____	Copy checked: _____

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EC REP CareFusion France 309 S.A.S.