Temno™

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



Temno, Carefusion and the Carefusion logo are trademarks or registered trademarks of CareFusion Corporation, or one of its affiliates.

© 2010, 2011, 2013, 2014, 2017, CareFusion Corporation or one of its affiliates. All rights reserved.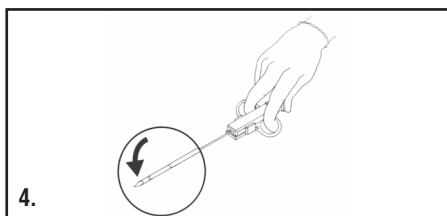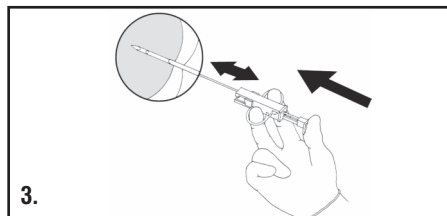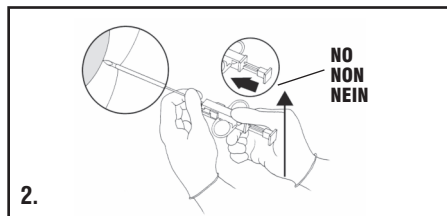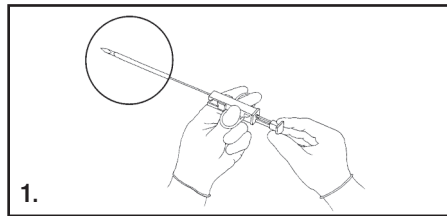

en

STEP 1
Charge Temno needle by pulling back hard on plunger until firm click is felt indicating that the needle spring is charged.

STEP 2
With stylet fully retracted (i.e. specimen notch completely covered by cannula) insert needle proximal to area to be sampled. **Warning: Be careful not to press plunger until needle is inserted to desired location.**

STEP 3
Press plunger with thumb to advance stylet exposing specimen notch. Press plunger past stop with thumb, automatically triggering the cannula to close, trapping the specimen in the sample notch.

STEP 4
Draw needle out. Pull back on plunger to recharge the spring. Push plunger to first stop exposing the specimen notch. Remove tissue specimen from notch in stylet. **Be careful not to push plunger past first stop during this step.**

Caution: For single use only. Re-use may result in a non-functional product or contribute to cross contamination.

Only clinicians who have been properly trained in this procedure, including patient preparation, sample preservation and the use of image guidance for percutaneous biopsy, should use this device.

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