

# Foam Rings for the SAVI<sup>®</sup> Applicator

## Instructions for Use

### DESCRIPTION

The foam rings for the SAVI Applicator are an absorbent pad placed near the proximal end of the catheters. The foam rings are provided non-sterile and are for single use only.

### INDICATIONS

The foam rings are intended for use with the SAVI applicator. The foam ring is capable of absorbing fluid and moisture.

### CONTRAINDICATIONS

The foam rings are not intended for use as a wound dressing.

### WARNINGS

The healthcare professional is responsible for its proper clinical use.

The foam rings are NOT a wound dressing and should not be in direct contact with the skin.

The foam rings are shipped non-sterile and **MUST NOT BE STERILIZED**.

The foam rings are for **SINGLE USE** only.

Do not use if the package is open or damaged.

### PLACEMENT OF FOAM RING

1. After the SAVI applicator has been placed, gently pull the peripheral catheters to the side of central shaft.\*
2. Using the slit in the foam ring, slide the ring over the central shaft just distal to the handle. (Fig. 1)

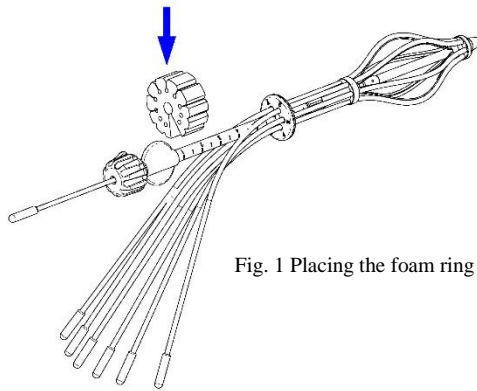


Fig. 1 Placing the foam ring over the central shaft

3. Carefully place each peripheral catheter into the appropriate slot of the foam ring. (Figure 2)

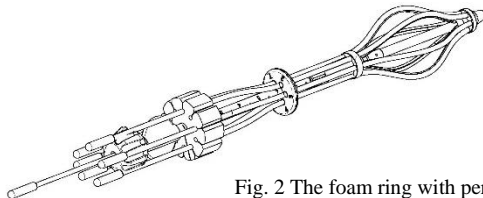


Fig. 2 The foam ring with peripheral catheters in place

### REMOVAL OF FOAM RING

When the foam ring has become saturated:

1. Carefully remove each of the peripheral catheters from the foam ring.\*
2. Using the slit in the foam ring, slide the ring over the central shaft removing it from the SAVI applicator.
3. Discard the foam ring.

\*Catheters protectors should be inserted in each catheter prior to any manipulation of the catheters.

### DISPOSAL PROCEDURES

It is the user's responsibility to dispose of all parts according to local regulations.

**STORAGE**

Store at room temperature. Avoid direct sunlight and storing the foam rings in conditions of excessive heat or humidity.

**COMPONENTS**

<u>Description</u>	<u>Quantity</u>
Foam Ring	10



Merit Medical Systems, Inc.  
6 Journey, Suite 125  
Aliso Viejo, CA 92656 USA