



# OR Quick Setup with SCOUT® Multi-Use Guide

## Console Overview



### 1 ATTACH GUIDE TO CONSOLE

- Align arrow on Guide connector with alignment notch on Console receptacle
- Press until secure connection made between Guide and Console


Alignment Notch

Alignment Arrow



### 2 POWER UP

- Turn on "Power" switch on back of console
- Once "SYSTEM READY" message appears
- Press "START/STOP" button to activate Guide
- "LOCALIZATION MODE" appears

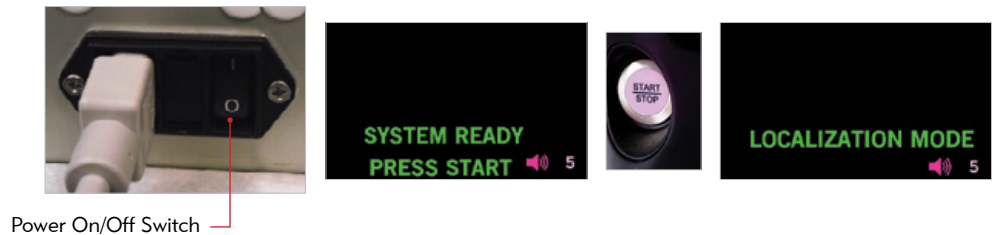

### 3 GUIDE TEST


- Before patient use, confirm system is in “LOCALIZATION MODE” and then place Guide on the Guide Test Card
  - Positive test – audible clicking, numerical readout increases
  - Negative test – no audible clicking, no increase in numerical readout. See below for troubleshooting
  - Pull sheath tightly over the Guide face to ensure no air gaps or folds on distal end
  - Secure by wrapping a steri-strip or sterile tape around the center of the Guide handle




**System is now ready to use**

## Console Display Messages


### Two most common modes

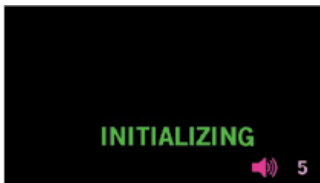


- Appears when console powered on
- ~30 seconds to “SYSTEM READY” mode

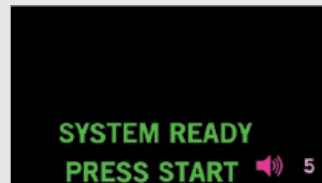

- Appears when system ready for use
- Mode for when Guide not in use

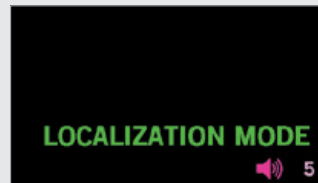

- Appears when “START/STOP” button pressed
- Guide active and ready to detect reflector
- Distance measurement appears once reflector is detected

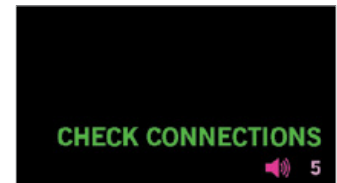

- Appears if issue with Guide or Guide connection
- Confirm secure connection
- Power off/on
- If message persists, replace Guide

## Troubleshooting if Negative Test on Guide Test Card

1. Power on/off
2. Secure Connection
3. Press Start/Stop to activate Guide
4. Retest using the Guide test card
5. If the problem persists, replace with new Guide

Before using refer to Instructions for Use for indications, contraindications, warnings, precautions, and directions for use. CE Mark pending.



merit.com  
ciannamedical.com

Merit Medical Systems, Inc.  
1600 West Merit Parkway  
South Jordan, Utah 84095  
Main: +1.801.253.1600  
Customer Service: +1.801.208.4300

Merit Medical Europe, Middle  
East, & Africa (EMEA)  
Amerikalaan 42, 6199 AE  
Maastricht-Airport  
The Netherlands  
+31 43 358 82 22

Merit Medical Ireland Ltd.  
Parkmore Business Park West  
Galway, Ireland  
+353 (0) 91 703 733