



SAVI SCOUT® Guide Sheath

SH-01

Instructions for Use

DESCRIPTION

The SAVI SCOUT® Guide Sheath is a component of the SAVI SCOUT® Surgical Guidance System. The sheath is a latex-free, sterile, single use cover for use with the SAVI SCOUT® Guide. The sheath allows for use of the Guide during the surgical procedure, while helping to prevent transfer of microorganisms, body fluids, and particulate material to the patient and health care worker during reuse of the Guide.

INDICATIONS FOR USE



The sheath is intended to be placed over the Guide during the surgical procedure.

WARNINGS

- In the USA, federal law restricts device to sale or use by or on order of a physician.
- Before use, you should be trained in using the SCOUT Guide. For instructions on the use of the Guide, please refer to the SCOUT Guide Instructions for Use.
- Dispose of single use components as infectious waste.
- See the SCOUT Guide Instructions for Use for cleaning of Guide between uses.
- Do not reuse, reprocess or resterilize single use device. Reuse, reprocessing or resterilization may create a risk of contamination of the Guide, cause patient infection or cross infection.
- Do NOT use if the package is open or damaged.
- Use the sheath prior to the expiry date shown on the product label.

RECOMMENDED PROCEDURE

1. Insert Guide into the sheath making sure to use proper sterile technique.
2. Pull sheath tightly over the Guide face to remove wrinkles and air bubbles, taking care to avoid puncturing the sheath.
3. Secure by wrapping a steri-strip or sterile tape around the center of the Guide handle.
4. Inspect sheath to ensure there are no holes or tears.
5. Following the procedure, remove the steri-strip or sterile tape prior to attempting to remove the sheath from the Guide.

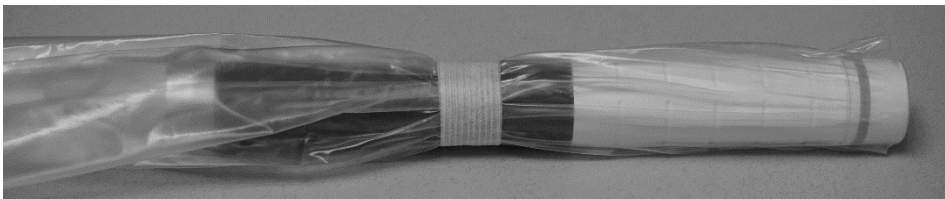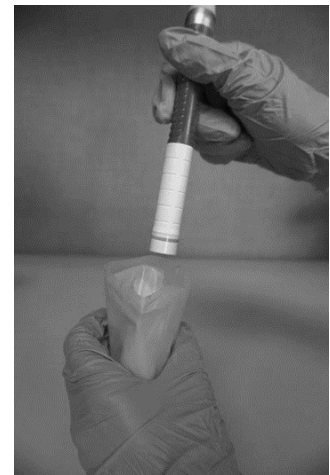

SYMBOLS USED ON LABELING



Consult Instructions for Use



Do Not Reuse



Do not use if package is damaged



Sterilized using Ethylene Oxide



Manufacturer



Expiry Date



Storage Temperature Range



Keep away from sunlight

MANUFACTURED FOR
Cianna Medical, Inc.
6 Journey, Suite 125
Aliso Viejo, CA 92656 USA
949.360.0059
949.297.4527 Fax

© 2019 Cianna Medical, Inc. All rights reserved. Patents and trademarks listed at:
<https://www.ciannamedical.com/patents>.