

SOLOPACE™ FUSION

Temporary LV Pacing System

Simplify LV Pacing

A next-generation solution, the SOLO PACE FUSION integrates left-ventricular (LV) pacing and valve delivery into one purpose-built guide wire package.

Replace Improvised LV Pacing Workarounds

The SOLO PACE FUSION system combines a specialized transcatheter heart valve (THV) delivery/pacing wire, a sterile single-use remote control module (RCM), a sterile patient connection cable, and a grounding pad to establish a stable, reliable pacing circuit without subcutaneous needles, alligator clips, or exposed metal surfaces.



KIT CONTENTS

FUSION Delivery/Pacing Wire

Outer Diameter	0.034" (0.858 mm)
Overall Length	275 cm
Shaft Coating	PTFE with proprietary insulative coating
Core Wire Material	304 stainless steel
Coil Material	304 stainless steel
Coil Coating	Silicone
Coil Height	3.0 cm
Coil Width	2.7 cm

Bluetooth-Enabled RCM

Allows operator control of the external pulse generator (EPG) from the sterile field

Integrated Patient Cable

Ensures proper polarity and connects the FUSION wire and ground pad to the EPG

Ground Pad

Serves as the anode return for unipolar LV pacing

Pair with the SOLO PACE CONTROL® EPG

The SOLO PACE CONTROL EPG (sold separately) provides customizable pacing profiles, operator-specific presets, automated capture-check, and backup pacing workflows. When used with the FUSION Kit, you can initiate, adjust, and terminate rapid LV pacing directly from the sterile field—minimizing communication delays and reducing the risk of pacing-related complications.¹



Intended Use

The SOLO PACE FUSION System is intended to introduce and position interventional devices within the chambers of the heart, including those used during transcatheter aortic valve replacement (TAVR) procedures. It may also be used to deliver temporary intracardiac pacing when connected to the SOLO PACE CONTROL EPG.

ORDERING INFORMATION

Catalog Number	Description	Components
SOLOFUSE1	SOLO PACE FUSION Kit	FUSION guide wire, RCM, integration cable, ground pad

Packaged 5 units per box

1. Data on File.

Before using, refer to Instructions for Use for indications, contraindications, warnings, precautions, and directions for use.



Understand. Innovate. Deliver.™

merit.com

Merit Medical Systems, Inc.
1600 West Merit Parkway
South Jordan, Utah 84095
1.801.253.1600
1.800.35.MERIT

Merit Medical Europe, Middle East & Africa (EMEA)
Amerikalaan 42, 6199 AE
Maastricht-Airport
The Netherlands
+31 43 358 82 22

Merit Medical Ireland Ltd.
Parkmore Business Park West
Galway, Ireland
+353 (0) 91 703 733